


# Scalable COVID-19 antigen testing in minutes

Get ahead of the spread with accessible testing for all


[siemens-healthineers.com/clinitest-covid-antigen](https://siemens-healthineers.com/clinitest-covid-antigen)

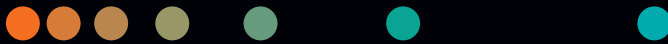

## CLINITEST Rapid COVID-19 Antigen Test

Now with easier specimen collection for improved accessibility

The lateral flow CLINITEST® Rapid COVID-19 Antigen Test,\* distributed by Siemens Healthineers, is visually read and easy to use, with results in just 15 minutes. No specialized laboratory personnel or equipment are required—the test can be administered anywhere, anytime by trained professionals. When used as part of a comprehensive strategy, the CLINITEST Rapid COVID-19 Antigen Test can help communities get ahead of the spread.

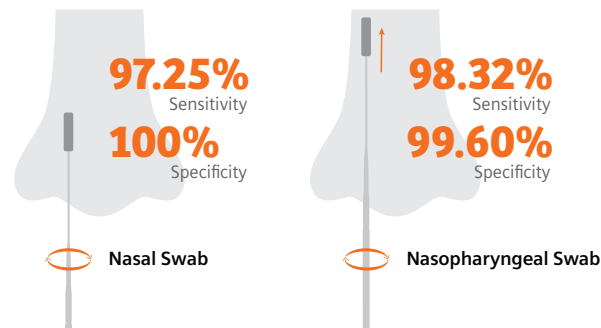

### **Rapid testing, reliable results**

Accurate results in only 15 minutes



### **Scalable testing for population health management**

Deployable for on-site testing where you need it most



### **Eliminates traditional barriers to access**

No lab personnel or equipment needed; two options for specimen collection

# Additional Product Information

Clinical performance of the CLINITEST Rapid COVID-19 Antigen Test was evaluated at seven sites within the U.S. where patients were enrolled and tested. Testing was conducted across several different sites, clinical environments, and operators. The resulting datasets for both nasopharyngeal and anterior nasal sampling comply with the latest requirements of most major regulatory authorities and demonstrate high-quality performance.

## Nasopharyngeal Swab Performance

Method	PCR (Nasopharyngeal Swab)		Total Results	
	Results	Positive		Negative
CLINITEST Rapid COVID-19 Antigen Test (Nasopharyngeal Swab)	Positive	117	3	120
	Negative	2	743	745
<b>Total</b>		<b>119</b>	<b>746</b>	<b>865</b>

Relative Sensitivity: 98.32% (95% CI:\* 94.06% to 99.80%)

Relative Specificity: 99.60% (95% CI:\* 98.83% to 99.92%)

Accuracy: 99.42% (95% CI:\* 98.66% to 99.81%)

\*Confidence interval

## Nasal Swab Performance

Method	PCR (Nasopharyngeal Swab)		Total Results	
	Results	Positive		Negative
CLINITEST Rapid COVID-19 Antigen Test (Nasal Swab)	Positive	106	0	106
	Negative	3	128	131
<b>Total</b>		<b>109</b>	<b>128</b>	<b>237</b>

Relative Sensitivity: 97.25% (95% CI:\* 92.17% to 99.43%)

Relative Specificity: 100% (95% CI:\* 97.16% to 100%)

Accuracy: 98.73% (95% CI:\* 96.35% to 99.74%)

\*Confidence interval

## Specifications

**Test time to result:** 15 minutes

**Sample type:** Nasopharyngeal swab or anterior nasal swab

**Storage:** Room temperature or refrigerated (2–30°C)

**Shelf life:** 24 months from date of manufacture

**Antigen detected:** Nucleocapsid

CLINITEST is a trademark of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product is under FDA review for Emergency Use Authorization (EUA) and not available for sale in the U.S. Product availability may vary by country. Distributed by Siemens Healthineers. Please contact your local representative for availability. For additional specifications and performance data, please refer to the instructions for use.

### Siemens Healthineers Headquarters

Siemens Healthcare GmbH  
Henkestr. 127  
91052 Erlangen, Germany  
Phone: +49 9131 84-0  
siemens-healthineers.com

### Published by

Siemens Healthcare Diagnostics Inc.  
Point of Care Diagnostics  
2 Edgewater Drive  
Norwood, MA 02062-4637  
USA  
Phone: +1 781-269-3000